Quick Start Guide

- FR** Petit Guide de démarrage
- NL** Snelstartgids
- DE** Schnellstartanleitung

Healthentia version 3.5

issue date: 08-01-2024

Innovation Sprint Sprl
Clos Chapelle-aux-Champs 30
Bte 1.30.30
1200 Brussels, Belgium


v3 (25/05/2021)

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NL 1 TOEPASSING DOWNLOADEN

DE 1 ANWENDUNG HERUNTERLADEN

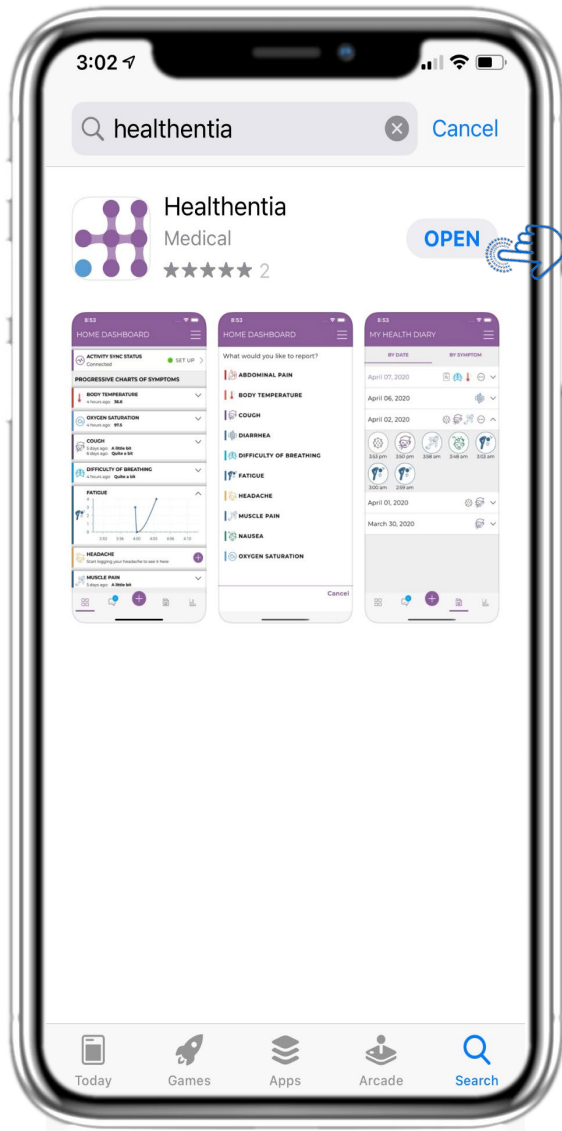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DE

2 ONBOARDING-FRAGEBOGEN

The screenshot shows a mobile app interface for entering weight and height. At the top, the time is 19:02. The screen is titled 'STEP 2/4' with a 'skip' button. The main heading is 'Enter your weight and height'. There are two input fields: 'Weight' with the value '63.5' and unit 'KG', and 'Height' with the value '1.75' and unit 'CM'. A purple 'Continue' button is below the fields. A disclaimer at the bottom reads: '*The information is used for selecting medical reference values for you for a more personalised experience.' A standard numeric keypad is visible at the bottom of the screen.

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ABOUT

The Healthentia platform, consisting of the Healthentia App and Portal, is considered as an active device. The platform consists of a collection of modules and most of them are non-medical module, there is however a module in the web application that complies with the definition of a medical device according to medical device directive 93/42/EEC. This module monitors non vital parameters to support decision making during clinical trial.

Per rule 12 of the Annex IX of the Directive 93/42/EEC, the HEALTHENTIA Platform is classified as a Class I Medical device.

For more information, please visit: <https://healthentia.com>

For any privacy related questions or requests, you can contact: dpo@healthentia.com

For general questions, you can contact: info@healthentia.com

Report of serious incident

Any serious incident that has occurred in relation to the Healthentia Medical Device App, you can contact the manufacturer (Innovation Sprint Sprl) in: support@healthentia.com and the authority having jurisdiction in your locale.

Technical support

Healthentia support service is offered at best-effort level.

For any technical support queries, you can contact: support@healthentia.com

Our support team will reply to you within 1 working day.

DISCLAIMER

Healthentia is a decision support tool, in which case the physician retains full responsibility for the medical decisions made.

The use of Healthentia is reserved for applications outside of critical situations (operating room, emergency, intensive care).

MINIMUM REQUIREMENTS

Supported hardware

Any smartphone that is using the supported Operating System (OS) – see below- is a supported hardware device.

Supported mobile OS

Healthentia App supports following operating systems in the Android and iOS platforms:

- Android: 6.0 and above
- iOS: 10.0 and above

Integration with other devices

Healthentia App is compatible with other devices to collect lifestyle information and vital signs.

The supported devices intended for use together with Healthentia are:

- Garmin trackers and watches via the Garmin API (Android & iOS)
- Fitbit trackers and watches via the Fitbit API (Android & iOS)
- iHealth devices via Bluetooth (Android & iOS)
- Any device that is integrated via iHealth Kit (iOS)

These devices constitute a safe combination and currently there is no device specific information on any known restrictions to combinations.

SECURITY & PERFORMANCE

Security

Innovation Sprint is committed to protecting the security of data subject's information and takes reasonable precautions to protect it, including protection against unauthorised access, necessary to run the software as intended. In this context, Innovation Sprint regularly evaluates and reviews technologies, facilities, procedures and potential risks to maintain the security and privacy of users' data, paying particular attention to role-based access and logging of health data access.

Moreover, any potential changes to software applications, provided services and/or hardware systems are properly justified, evaluated and registered based on international best practices and standards.

Performance

Healthentia is a Class I medical device software intended for Software intended for monitoring of non-vital parameters to support decision making and virtual coaching of patients during clinical trials or under a medical or wellbeing treatment context.

Given the absence of direct interaction with the human body, and therefore the independence of the performance from the user, the clinical performance of the device depends only on its technical performance.